



Sterimatic Worldwide Ltd.

Sterimatic Worldwide Ltd.

Griffin Mill, London Road, Stroud, Glos, GL5 2AZ, UK

Tel: +44(0) 1453884944 Fax: +44(0) 1453886481

Mobile +44 (0) 7973361593 Vat Reg: 800 8086 54

Registered Office: Greenmarley, Pinfarthings, Stroud, Glos, GL5 5JQ. Reg. No.: 4592319

E-mail: prowley@sterimatic.com Web-site: www.sterimatic.com

Option 1: Syringe (SYX045)+ Sterimatic system (push fit sleeve)

Syringe + Green sleeve (Push fit) + Stericap



- Attach the needle.
- Push the sleeve onto the syringe luer fitting.
- Insert Stericap into sleeve and remove twist tab



- Now inject as normal.
- As the needle is inserted into the animal the sleeve retracts and the Stericap wipes the needle.
- Deliver the dose.
- As the needle is withdrawn it is automatically cleaned and protected.





Sterimatic Worldwide Ltd.

Sterimatic Worldwide Ltd.

Griffin Mill, London Road, Stroud, Glos, GL5 2AZ, UK

Tel: +44(0) 1453884944 Fax: +44(0) 1453886481

Mobile +44 (0) 7973361593 Vat Reg: 800 8086 54

Registered Office: Greenmarley, Pinfarthings, Stroud, Glos, GL5 5JQ. Reg. No.: 4592319

E-mail: prowley@sterimatic.com Web-site: www.sterimatic.com

Option 2: Syringe (SYX045) + Sterimatic system (Adapter (ANA012) + Screw-fit sleeve)

Syringe + Adapter +
Green sleeve (Screw-fit) + Stericap



- Remove the Luer-fitting from the syringe and attach the adapter and re-fit the luer-fitting.
- Attach the needle.



- Push the sleeve onto the syringe luer fitting.
- Insert Stericap into sleeve and remove twist tab



- Now inject as normal.
- As the needle is inserted into the animal the sleeve retracts and the Stericap wipes the needle.
- Deliver the dose.
- As the needle is withdrawn it is automatically cleaned and protected.

